



MERZ AESTHETICS

Beauty true to  
every side of you.

Information for Patients


[www.merzaesthetics.co.uk](http://www.merzaesthetics.co.uk)

BELOTERO® VOLUME

## How age affects your appearance

Nothing influences our appearance more than age. It's no secret that fine lines and wrinkles appear later in life. Loss of elasticity, as our collagen and elastin levels go down, along with changing bone structure and the downward shift of fat beneath the skin, also transform our faces. Sometimes as we grow older, we are looking for beautification, with added volume and projection.

If you want a minimally invasive solution that serves your beauty from every angle, BELOTERO® Volume can meet your beauty needs at any age.

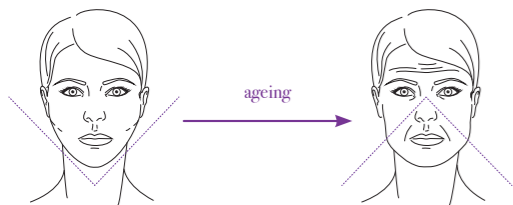

### The look you'd like to preserve

A more youthful appearance is always valued. You can help achieve this by enhancing the volume in different areas of your face. This also contributes to a more balanced and harmonious facial shape for people at any age.

### What you might be fearing


BELOTERO® Volume naturally integrates into the tissue so you can retain your identity and express your emotion with natural-looking results. You might be wary of cheeks with the appearance of implants, not to mention pain or fear of swelling and bruising.

To find out how BELOTERO® provides a more youthful appearance that naturally blends with your own, read on!

# What is BELOTERO® Volume?

Facial ageing occurs due to reduced facial fat, loss of bone support and destruction of collagen and elastin. The key ingredient in BELOTERO® Volume is hyaluronic acid, which helps support collagen and elastin, restores lost fat volume and mimics bone support.

As an injectable dermal filler, BELOTERO® Volume works beneath the skin, actively reducing the signs of ageing - leaving you with results that speak volumes.

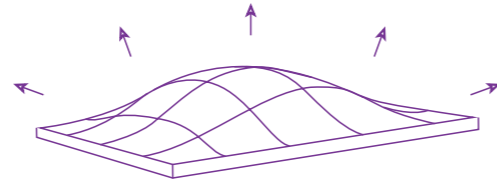

## Where BELOTERO® Volume works best

Our volumiser is ideal for restoring volume and restructuring areas such as the chin and the cheeks.

## How BELOTERO® Volume stands out

Proven to be long lasting, our volumiser is based on our unique 3D-Volume-effect. This means it volumises by creating areas of high and low density which allows optimal tissue integration and a very natural feel.

## What's more


- Once injected, the gel can be shaped and moulded to bring harmony to the face
- Enables smooth transitions on your face between treated and untreated areas
- Blends in with your facial features for a natural looking result

## And that's not all

BELOTERO® Volume brings excellent volumising results, durability, and satisfaction: just what you desire and deserve.

LASTS UP TO

**18**  
months<sup>6</sup>

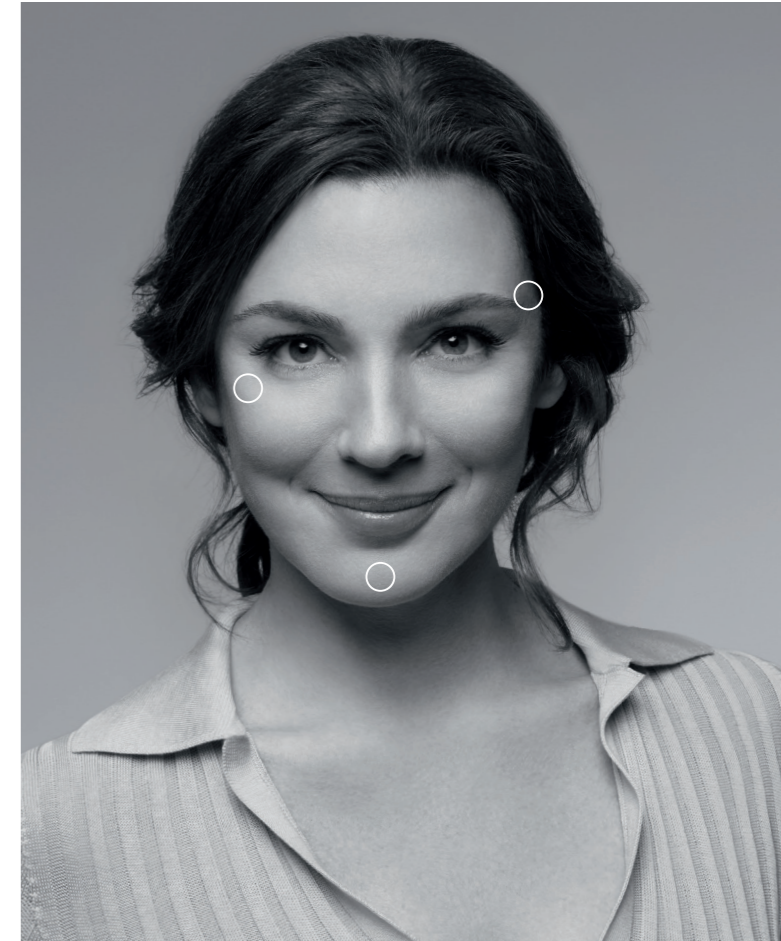

Camille, 42, actual BELOTERO® Volume patient

# Real and natural looking results

BELOTERO® respects every patient's individual needs and unique beauty.

BELOTERO® Volume can correct imbalances, improve contours and restore volume in multiple areas of the face/or in the upper AND lower face.

Here are some individual success stories:

Gwen aged 40  
Treatment with  
BELOTERO® Volume in  
temples and cheeks.



Jure aged 40  
Treatment with  
BELOTERO® Volume in  
chin and lower face.



Martina aged 32  
Treatment with  
BELOTERO® Volume in  
temples, cheeks, chin  
and lower face.



Before



2 weeks after injection



Individual results may vary

Individual results may vary

Individual results may vary

# How to get your treatment: Step by step

Only an individual treatment plan can bring out the best in your appearance. Here's how to start your journey today:

- 1 Book your appointment**  
It's natural to be a little nervous, feel free to ask questions and have all your concerns addressed.
- 2 Attend your consultation**  
Get a unique treatment plan based on your needs. Be sure to tell your aesthetic practitioner of any allergies you have or medication you are taking.
- 3 Treatment lasts less than one hour**  
You can return to your day immediately after treatment, but we recommend first resting a little.
- 4 Ask for specific after-care advice**  
No dedicated recovery time is needed, but avoid direct sun, sauna visits and intensive sport sessions.
- 5 Admire your look in the mirror**  
Enjoy beauty true to every side of you!



“95% of BELOTERO® Volume patients were satisfied with their results for up to 18 months and would repeat the treatment and recommend it to their friends”



## Discover the Merz family

As creator of BELOTERO® Volume, Merz has achieved long-standing success in the field of facial aesthetics. Here's more about our mission and our full range of products.

### The BELOTERO® range

BELOTERO® Volume is part of a versatile product family of hyaluronic acid fillers. The BELOTERO® range is versatile enough to address a range of concerns including lip volumisation, softening fine lines, smoothing contours and restoring structure.

### Merz Aesthetics

To us, aesthetics is more than just improving looks. We believe each patient has their own unique beauty, and we treat them that way. With over a century of experience, we firmly believe innovation and trusted relationships provide our patients with a unique treatment solution. Ask your aesthetic practitioner for more information or visit us at:

[www.merzaesthetics.co.uk](http://www.merzaesthetics.co.uk)



Merz Pharma UK Ltd.  
260 Centennial Park, Elstree Hill South,  
Elstree, Hertfordshire, WD6 3SR  
Tel: +44 (0) 333 200 4140

BELOTERO® is a registered trademark of  
Merz Pharma GmbH & Co. KGaA.

Please consult a qualified authorised medical practitioner  
who will advise the most appropriate treatment for you.  
Individual results may vary.

M-BEL-UKI-0793 Date of Preparation June 2020